

activ'm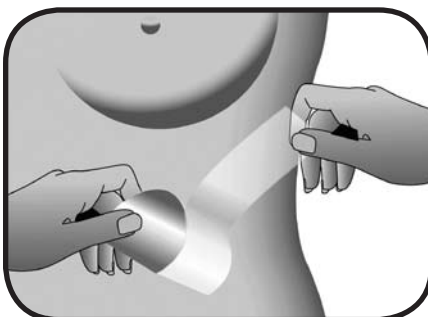

Gebruiksaanwijzing Mode d'emploi Instructions for use Gebrauchsanweisung

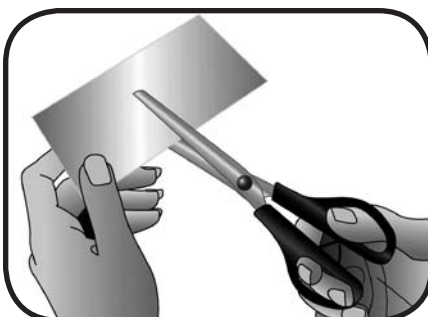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BAP Medical BV
P.O. box 531, 7300 AM Apeldoorn
Pays-Bas

ENG — Instructions for use

Activ'M is a self-adhesive silicone dressing made from medical-grade silicone and a flexible polyurethane film. These components have been selected because of their biocompatibility. Activ'M is so flexible that the silicone dressing is easy to use on all parts of the body, including under clothing.

Indications for Activ'M

Silicone dressings have a positive effect on:

- the appearance (height) of old and new scars
- the treatment of hypertrophic and keloid scars
- the prevention of hypertrophic and keloid scars
- the treatment of burns and donor sites

Contra indications of Activ'M

Activ'M cannot be used on:

- open wounds
- macerated skin

How to use Activ'M silicone dressings

Building up the therapy

You should gradually build up the use of Activ'M during the first week of use, until you are wearing the dressing 24 hours a day if necessary. This approach helps your skin get used to the effect of a silicone dressing.

The results of the therapy depend on how long each day you can wear Activ'M. The longer you wear it per day, the better the results will be.

Before using Activ'M

Clean the skin around the scar very thoroughly and make sure it is not greasy.

Cut the silicone dressing to fit. Make sure that the whole scar and approximately 2 cm of healthy skin around the scar are covered.

Applying Activ'M

Remove the first part of the blue protective

cover of the silicone dressing and place the self-adhesive silicone dressing carefully at the edge of the scar.

Remove the second part of the blue protective cover of the silicone dressing and press it carefully on the scar.

Depending on the size of the silicone dressing you are using, you may have to repeat this process until the entire silicone dressing is in place.

Activ'M does not need any other product to help it stick.

Cleaning Activ'M

Activ'M is a single use silicone dressing that you can leave in place for several days.

After using Activ'M

We advise you to moisturise the skin, for example with ALHYDRAN medical after care gel-crème, when you remove the silicone dressing.

Replace the silicone dressing with a fresh silicone dressing as soon as its adhesive properties start to decline.

How long do Activ'M dressings last?

Each Activ'M pack contains 10 silicone dressings and is sufficient for 4 to almost 8 weeks of therapy. Each silicone dressing should be used for no more than 5 to 7 days at the most.

Warning

Consult your doctor if you experience skin irritation or skin problems

Activ'M silicone dressings are for external use only

Use Activ'M silicone dressings only on closed skin and never on open wounds

Exclusive distribution for Benelux

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