BAPSCARCARE T

silicone therapy for scars

Instructions for use of BAP SCAR CARE T

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ENG

BAP SCAR CARE T silicone dressing

Self-adhesive, thin and transparent medical silicone dressing for the treatment and prevention of abnormal (hypertrophic and keloid) scars.

Product description:

BAP SCAR CARE T is a range of silicone dressings for the treatment of both old and new scars and more extensive scars. Because of its flexibility, the BAP SCAR CARE T self-adhesive dressing stays in place on any part of the body. Suitable for use under clothes.

Properties:

BAP SCAR CARE T silicone dressings make scars flatter, more supple and thinner and help reduce symptoms (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints). The dressing provides a covering, occlusive environment on top of the skin, resulting in increased hydration of the skin (epidermis). The increased hydration helps the damaged scar tissue heal more quickly and reduces functional complaints.

Indications/uses:

BAP SCAR CARE T silicone dressings can be used on both old and new scars:

- Treatment and prevention of abnormal scars (hypertrophic and keloid)
- Treatment of burn scars
- Treatment of surgical scars
- Persistent functional complaints (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints)

How to use:

Building up the treatment

We recommend building up the use of the treatment to 24 hours a day during the first week to allow the skin to get used to the treatment gradually. For the best possible result, the silicone dressing should be worn for as long as possible each day.

Before use

Clean and dry the skin on and around the scar thoroughly. Remove the product from the pack. If necessary, the silicone dressing can be cut to size. Make sure the dressing covers the entire scar and approx. 2 cm of the healthy skin around it.

Application

Remove the first part of the protective film on the silicone dressing and apply the self-adhesive silicone dressing carefully to the beginning of the scar.

Remove the second part of the protective film and apply the next part of the silicone dressing to the scar. Depending on the size of the dressing, repeat the above until the entire dressing has been applied. The silicone dressing does not need to be specially secured.

Maintenance

The BAP SCAR CARE T silicone dressing is a single-use dressing which you can leave on the scar for several days. Change the silicone dressing when its adhesive grip starts to wear off.

How long can I go on using BAP SCAR CARE T?

You can use BAP SCAR CARE T for as long as the scar is raised, red or gives rise to symptoms. Ask your doctor for a precise treatment plan.

Duration of use

Each pack of BAP SCAR CARE T contains 10 silicone dressings, each of which remains usable for between 5 and 7 days. A full box of BAP SCAR CARE T is sufficient for between 5 and 8 weeks of treatment.

Precautions

- Consult your doctor in the event of skin irritations or skin problems.
- The silicone dressings are for external use only.
- Only use the silicone dressings on closed skin: never use on open wounds.

Side-effects:

No side-effects have been reported to date.

Storage:

See pack for expiry date. Store at room temperature, but not exceeding 25°C.

BAP SCAR CARE silicone treatment for scars is available as:

- an ultra-thin (0.3 mm) and transparent silicone dressing (BAP SCAR CARE T),
- a thick silicone dressing (1.4 mm) (BAP SCAR CARE S) and
- a BAP SCAR CARE silicone gel, enriched with vitamin E.

Date of last revision of patient information sheet:

November 2010

Legal Manufacturer: BAP Medical BV, Laan van de Maagd 119, 7324 BT Apeldoorn, The Netherlands. Produced exclusively for BAP Medical by M-Technologies, France.

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