

# Case ALHYDRAN: Second degree burn

## 1. Case study description

patient age	45 years
gender	male
cause	fireworks burn, 31th of December 2006
treatment to wound closure	Flammazine
therapy after care	5 weeks post accident start with ALHYDRAN, 3 daily doses, brought on thick during the first few weeks

## 2. Evolution of the wound



## 3. Conclusions medical team

- Deep second degree burn with serious ankle damage
- Therapy starts with Flammazine, patient evolves well, only a lot of complaints on itching of closed skin
- After 5 weeks: start with ALHYDRAN on the freshly closed skin, 3 times per day
- ALHYDRAN had good result on the skin, which improved rapidly
- Itching complaints disappeared immediately when ALHYDRAN was used

## 4. Patient experience

- Flammazine gave a lot of itching problems
- Upon start of the treatment with ALHYDRAN, immediate reduction of itching
- Therapy: 3 daily doses, or each time the skin was itching
- After 6 months, the skin was soft and calm, patient started sporting again
- Skin looks good and calm now
- After 18 months, maintenance of 1 daily dose of ALHYDRAN

Case study: Dr. K. Veel MD, AAB Aruba